



Ultra Gloves

(Powder Free Examination Gloves)



Code No.: 9016

1- Description:

- The product is used to provide the best barrier protection in areas of potential risk associated with blood-borne pathogens and other substances or biohazards.

2- Advantages & Features:

- Excellent flexibility and uncompromised protection with high degree of sensitivity.
- Softness provides superior comfort and natural fit
- Ambidextrous and straight fingers.
- Easy donning and helps prevent roll back
- Powder free and non-sterile.

3- Indications:

- To cover hands during & protection in areas of potential risk associated with blood-borne pathogens and other substances or biohazards.

4- Contraindications:

- Not to be used in patients with known hypersensitivity to any of the materials used.

5- The purpose:

- The product is used to provide the best barrier protection in areas of potential risk associated with blood-borne pathogens and other substances or biohazards.
- For single use.

6- Instruction for Use:

- Wash your hands before donning gloves whenever possible. Pull a glove from the box with one hand. Most people start by holding the glove in their non-dominant hand.

- Holding the glove by the open edge with the fingers of your non-dominant hand, make sure that the glove is facing the right direction to facilitate the thumb without fumbling.
- Slide your opposite hand into the base of the glove, fingers spread slightly apart. Make sure that you find each opening with the tips of your fingers before you push your hand all the way in. Wriggle your fingers slightly to make sure that the glove is seated firmly and comfortably.
- Pull the other glove from the box and repeat the process with the other hand.
- Examine the gloves to make sure that there are no holes or tears prior to contact with a patient or any surface.

7- Care instructions:

- Single use only.

8- Storage conditions:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Suitable temperature (18 – 32 C).



9- Product code with size and measures:

- Code No.: 9016
- Sizes : Small , Medium , Large , X-Large

Code	Size
9016-1	Small
9016-2	Medium
9016-3	Large
9016-4	X-Large

10- List of product components:

- Natural high quality rubber latex Powder free

11-Shelf life: 5 Years.

Assiut Factory: From Part No. 304 to Part No. 310 and Part No. 312, Arab El Awamer- industrial zone, Abnoub, Assiut, Egypt.. **Tel:** 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

& Fax:002-088-4964222**& Mob:** 002-01001558853, 002-01068832355

Cairo Head Office: 64, Nakhla El Motaiy Triumph Square Heliopolis **Tel:** 022/4171621-4143794 **& Fax:** 022/4171613 **& Mob:** 01223988200 **Assuit Office:** 23, July Str. **Tel:** 088/2364111 – 2364222 **& Fax:** 088/2334964 **& Mob. :** 01223988202

Alexandria Office: 212Abd El salaam Airef Str, Luran **Tel:** 03/5856202– 5856458 **& Fax:** 03/5828988**& Mob:** 01223948666

E-mail: shady@elaggargroup.com **Website:** www.ultramedumic.com